

The bloodless castrator, Burdizzo pattern, for horses, bulls, sheep, pigs, etc. : instructions for use ... prices ... / Arnold & Sons.

Contributors

Arnold & Sons.

Publication/Creation

London : Arnold & Sons, [ca.1930].

Persistent URL

<https://wellcomecollection.org/works/htdjc8tg>


License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

THE
BLOODLESS CASTRATOR
BURDIZZO PATTERN
FOR
Horses, Bulls, Sheep, Pigs, etc.

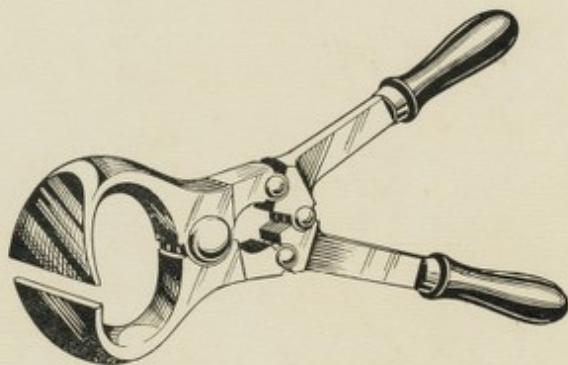

For prices see overleaf.

INSTRUCTIONS FOR USE.

Secure the animal firmly by the head to a post or any substantial support.

Bulls should be operated on standing. Horses, sheep and pigs, are preferably thrown. With smaller animals, it is sufficient to secure the services of an assistant, to hold the animal in his arms.

Standing behind the animal, the Operator should pull one testicle well down into the bag. Holding the instrument, with the jaws ready open in one hand, the Spermatic Cord can be located, and held against the side of the bag. Where the cord is held, place the jaws of the instrument. Close the instrument against the cord a little above the testicle. Having ensured that the cord is gripped, press the handles with both hands hard until the jaws are completely closed.

A second compression should be made immediately on the same cords a little higher, to crush the cord in two distinct places.


The operation is then repeated, in exactly the same manner with the other cord, when castration is complete.

IMPORTANT.—Make certain that the cord is gripped in the jaws of the instrument otherwise the operation will be ineffective. The second crushing on the same cord is to ensure that the severance of each cord is successfully carried out.

The testicles will shrink and become useless, the process taking from four to six weeks. There is no bleeding, the outer skin is not cut, and the animal will not lose condition, or need further treatment.

(P.T.O.)

THE BLOODLESS CASTRATOR is also made in smaller sizes, suitable for lambs, small dogs, etc.

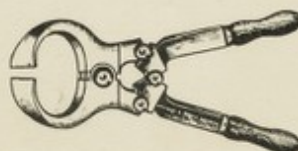

BABY SIZE.

Baby size, as illustrated, for prices see below.

The Operation on these is quickly performed, in the manner indicated overleaf. The animal is usually firmly held in a convenient position by an assistant, whilst the operator performs the same details, as for the larger animals.

PRICES.

Burdizzo, Original, Nickel-plated	...	in two sizes—large and small	£3 15s. 0d.	3.0.0.
		Postage 1/-		
Burdizzo Pattern (Continental Make)	...	Large size	£2 15s. 0d.	2.14.0.
		Postage 1/-		
Do. do. do.	...	Medium size	£2 10s. 0d.	2.6.6.
		Postage 1/-		
Do. do. do.	...	Small Size	£2 5s. 0d.	2.4.0.
		Postage 9d.		
Do. do. do.	...	Baby size	£1 7s. 6d.	1.6.6.
		Postage 9d.		

ARNOLD & SONS

(JOHN BELL & CROYDEN)

Makers of Veterinary Instruments and Appliances.

50-52, WIGMORE STREET, LONDON, W. 1.

Works: Lawrence Road, Tottenham, N. 16.

Telegrams :
Instruments, Wesdo, London.

Telephone :
Welbeck 5555 (10 lines).