

The castration of lambs : from one week to six months with the new extra-small model, the Baby "Burdizzo" registered designs.

Contributors

N. Burdizzo (Firm)

Publication/Creation

[Turin? :N. Burdizzo, 1928?]

Persistent URL

<https://wellcomecollection.org/works/samzubhq>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

The Castration of Lambs

... FROM ONE WEEK TO SIX MONTHS ...
WITH THE NEW EXTRA-SMALL MODEL, THE:
BABY "BURDIZZO"
REGISTERED DESIGNS

Hold the BABY "BURDIZZO", opened as indicated in fig. 1,

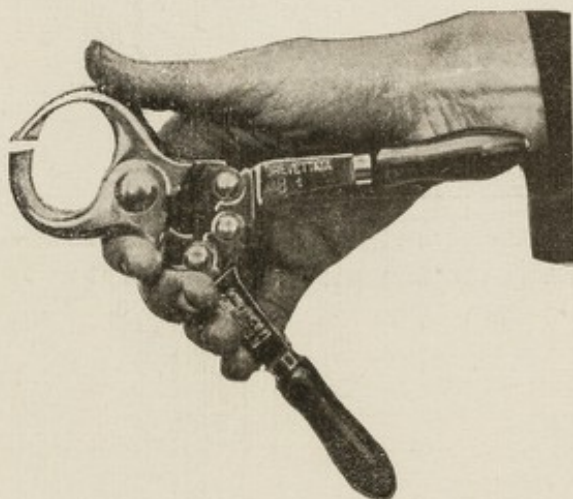


Fig. 1.

and apply its jaws on to the cord of the left testicle. The lamb should be held by an attendant as indicated in fig. 2 or in fig. 5 according to its size.



Fig. 2.

All rights of total or partial transcription or reproduction are reserved under the international copyright act.

Printed in Italy

«La Grafica», Turin [104], December 1928.



Fig. 3. — Enlarged detail of fig. 2, showing how the bag should be grasped for retaining the cord ready and firm under the crush of the Pincers.

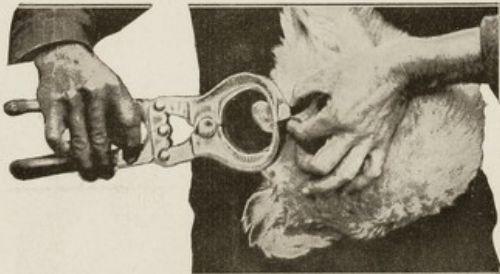


Fig. 4. — When the Pincers are completely closed, press the cord immediately above the jaws, with thumb and second finger until these almost meet, so indicating that the cord is completely severed. Open the Pincers and repeat the operation on the cord of the right testicle.

The Castration of RAMS of any age with the Small Size "Burdizzo,"

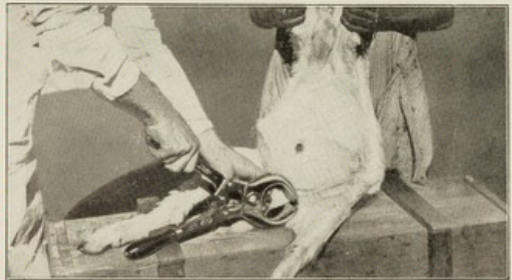


Fig. 5. — Seat the ram on his haunches. The operator holds the cord in place, as indicated in fig. 3, with one hand and with the other applies the open Pincers, holding them by the upper handle. He then presses the other handle on the support where the ram is seated until the Pincers are fully closed, and severs the cord with the fingers, as shown by fig. 4.

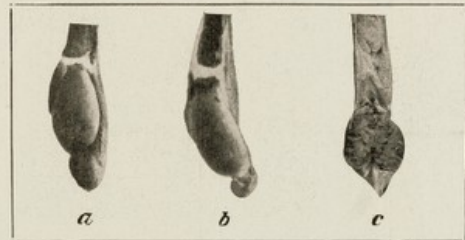


Fig. 6.

a b — Actual photographs of lambs' testicles immediately after the «Burdizzo» operation. The cord is seen to be completely severed while the sac WAS NOT CUT.

c — Result of the «Burdizzo» operation on a lamb six months old which was castrated at two months. The testicle is reduced to a lump of fat, the size of a coffee bean.

The only "BURDIZZO" is the one
 invented and made by Dr. Burdizzo,
 Turin, Italy.
 BURDIZZO
 LA MORRA ITALIANA

DOCKING OF LAMBS.

The Lamb should be held as shown in Fig. 2 or Fig.



Fig. 7 — Close the SMALL SIZE « BURDIZZO » on tail — Cut tail off inside the closed jaws—Treat the cut with coal tar—Release the lamb—Crush of Pincers will prevent all bleeding.

Results of the «BURDIZZO,, Way. (CASTRATION AND DOCKING)

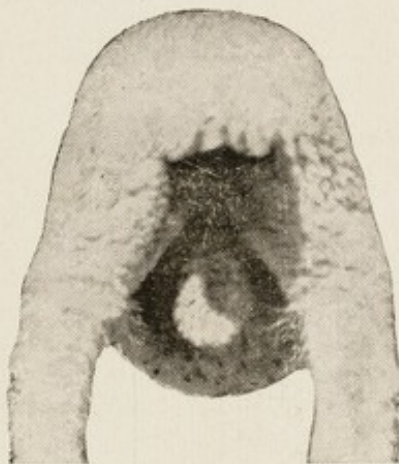


Fig. 8.

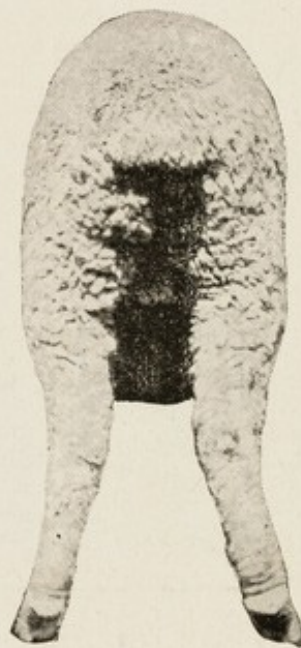


Fig. 9.

Fig. 8. — IMMEDIATELY AFTER THE OPERATION. Actual photograph of young lamb showing that the bag was not cut or otherwise damaged and the tail did not bleed. After the first squeeze the lamb seems to take no further notice of the operation.

Fig. 9. — A WETHER who was castrated and docked when two months old. The bag is reduced to a handful of fat, which provides the « talking point » for the best market prices.



N. BURDIZZO
LA MORRA (ITALIA)

Pincers not stamped with these
Trade Marks are not «BURDIZZO»
Beware of imitations