

R. L. Lewis pattern cord-holding forceps : prov. pat. 1657 : for use in conjunction with the "Burdizzo" (or similar) castrator / Arnold & Sons.

Contributors

Arnold & Sons.

Publication/Creation

London : Arnold & Sons, [192-?]

Persistent URL

<https://wellcomecollection.org/works/zgfva2hy>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

R. L. LEWIS PATTERN


CORD-HOLDING FORCEPS

Prov. Pat. 1657.

FOR USE

in conjunction with

THE "BURDIZZO" (or similar) CASTRATOR.

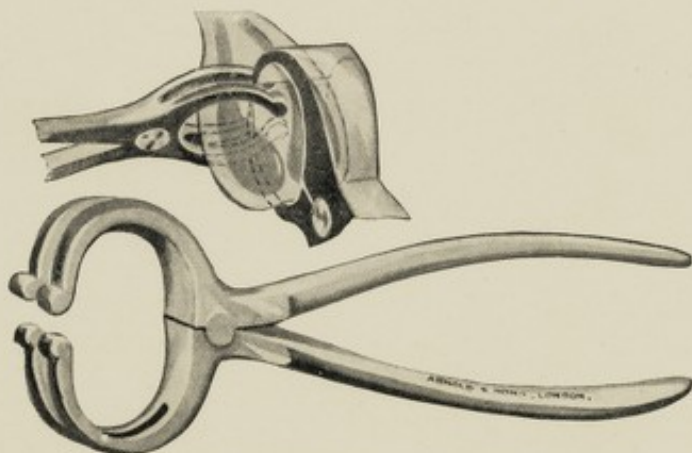

CORD-HOLDING FORCEPS (R. L. Lewis) Prov. Pat. 1657.

These Forceps have been devised as a result of considerable experience in the use of the "Burdizzo" Castrator, and with a view to rendering the latter foolproof, and the operation of castration certain.

The object is to prevent escape of the Spermatic Cord from the jaws of the castrator, which does not infrequently occur, and thereby necessitating a second operation.

The inventor claims that escape of the cord is impossible, and that, in most cases, it is quite unnecessary to crush the cord in more than one position, the forceps aiding the complete severance of the Spermatic vessels at one squeeze of the pincers.

Can be used on ANY Animal upon which the "Burdizzo," or any similar castrator, is used, either in the standing or recumbent position.

INSTRUCTIONS FOR USE.

The Forceps are placed in position from the left side, gripping the scrotum between the two spermatic cords. The "Burdizzo" or any similar castrator, is then applied as in the usual method of castration, but in the space provided between the bifurcated jaws of the forceps. The operation on the right spermatic cord is completed in the ordinary way by closing the jaws of the castrator. The operation is performed in the same manner on the left spermatic cord by changing the position of the forceps, that is, by fixing it between the two cords from the right side. The forceps are held in position by an assistant.

Black Finished Price **20/-** Each.
Nickel-plated 22/6 each

ARNOLD & SONS,

(Savory & Moore, Ltd.)

Makers of Veterinary Instruments and Appliances,

50/52, WIGMORE STREET, LONDON, W.1.

Works: Lawrence Road, Tottenham, N. 15.

Telegrams :
Instruments, Wesdo, London.

Telephone :
Welbeck 5555 (10 lines).