

Intersep urine concentrator / [Intersep Filtration Systems].

Contributors

Intersep Filtration Systems.

Publication/Creation

[Wokingham] : [Intersep Filtration Systems], [1993?]

Persistent URL

<https://wellcomecollection.org/works/npwxnyam>

License and attribution


Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).




Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

INTERSEP URINE CONCENTRATOR

1. Pipette 4.04ml
urine into
reservoir

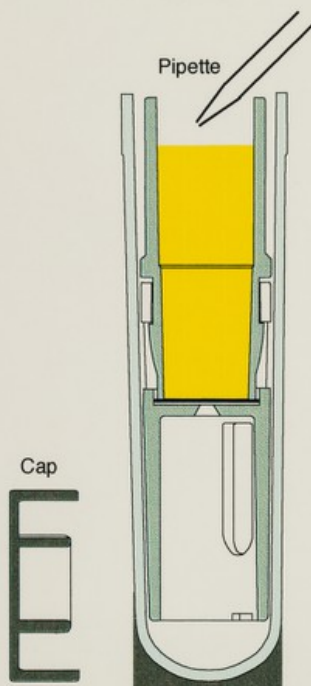

2. Cap unit and
centrifuge at
4000g for 120
mins

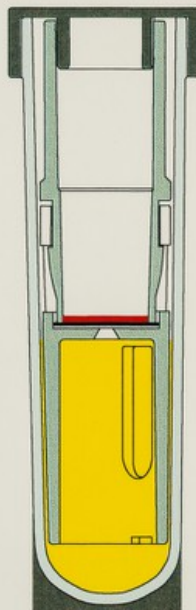

3. Remove 0.04ml
concentrate
Remaining 4ml is
filtrate

