

Alcon Softlab : for the disinfection of all soft contact lenses, preservative free, requires no neutralisation.

Contributors

Alcon Laboratories (UK) Ltd

Publication/Creation

Watford : Alcon Laboratories (UK) Ltd, [approximately 1990]

Persistent URL

<https://wellcomecollection.org/works/kd2rfxzg>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

1111/2

K204161801



Alcon

For the disinfection of
all soft contact lenses
Preservative free
Requires no neutralisation

32 TABLETS

Softab[®]

Batch No:

Contents: 32 tablets individually sealed in foil
Tablets for use in the daily disinfection of soft (hydrophilic) contact lenses. Each tablet contains: sodium dichloroisocyanurate 0.065mg calculated as available chlorine in an effervescent base.

Directions for use

Wash hands thoroughly. Clean each lens with PLIAGEL and rinse thoroughly with SALETTE or ALCON AEROSOL SALINE to remove surface contamination and many micro-organisms. Place the lenses in the SOFTAB case, filled to the line with SALETTE or ALCON AEROSOL SALINE. Add one SOFTAB tablet, close the case and allow to stand overnight or for a minimum of four hours. At the end of the soaking period rinse the lenses thoroughly with SALETTE or ALCON AEROSOL

SALINE. Your lenses are now ready for wear.

Use only SALETTE, ALCON AEROSOL SALINE or other physiological sterile saline. Do not use SOFTAB with preserved salines or soaking solutions. SOFTAB should not be dissolved in distilled water or tap water.

DO NOT MIX THE TABLETS OR SOLUTIONS WITH OTHER FLUIDS EXCEPT AS RECOMMENDED. NEITHER THE TABLETS NOR THE SOLUTION SHOULD BE APPLIED DIRECTLY TO THE EYE.

Not to be taken internally. Keep out of reach of children.

If you experience eye irritation or any other adverse reaction consult your doctor or optical practitioner.

PL 0649/0095

Alcon Alcon Laboratories (UK) Ltd., Imperial Way, Watford, Herts. WD2 4YR

Expiry Date:

331C



14