

Labstix.

Contributors

Ames Company

Publication/Creation

Slough : Ames Company, [1967?]

Persistent URL

<https://wellcomecollection.org/works/zeg56yqp>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

650 2

June '93 Dr.Emily V. Saunders-Jacobs : Orpington, Kent.

Labstix

TRADE MARK


REAGENT STRIPS

1. Dip test areas of strip in fresh, well-mixed urine.
2. Gently tap edge of strip to remove excess urine.
3. Compare colour of test areas with corresponding colour chart at the time specified.

(Read in order of arrow)



Ames

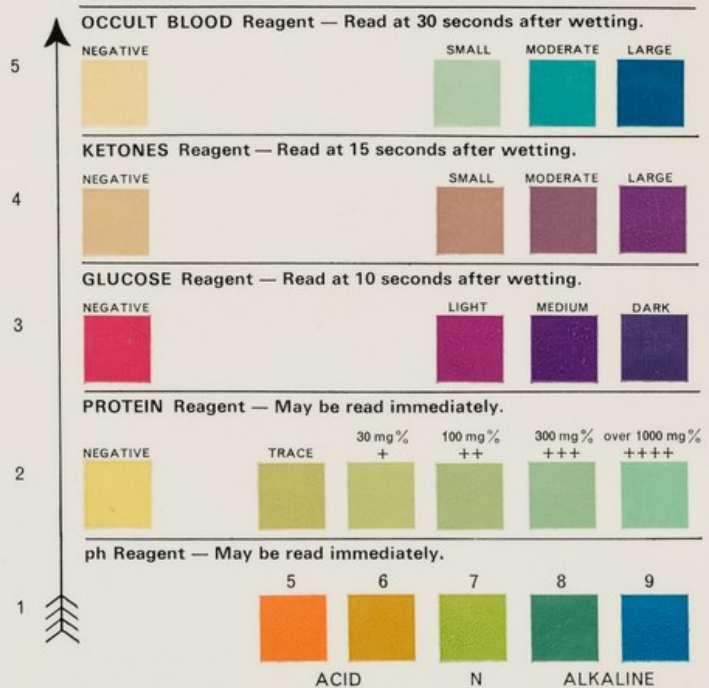

Ames Company Division of Miles Laboratories Ltd.
Stoke Poges, Slough, Bucks, England

PRINTED IN GREAT BRITAIN

HQ6706

**NOW COMPLETE YOUR URINE EXAMINATION BY USING
ICTOTEST for BILIRUBIN**

1. Place 5 drops of urine on square of special test mat.
2. Place Ictotest tablet in middle of moist area.
3. Flow 2 drops of water over tablet.
4. Observe colour of mat around tablet exactly 30 seconds later.

Positive Mat around tablet turns bluish-purple within 30 seconds. Amount of bilirubin is roughly proportional to intensity of bluish-purple colour and speed with which it develops.

Negative Mat around tablet remains unchanged at 30 seconds or turns faintly red or orange.