

**A guide to condoms : the Durex range / Durex Information Service for Sexual Health.**

**Contributors**

Durex Information Service for Sexual Health

**Publication/Creation**

[between 1995 and 2005?]

**Persistent URL**

<https://wellcomecollection.org/works/kg76a5z4>

**License and attribution**

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).



Wellcome Collection  
183 Euston Road  
London NW1 2BE UK  
T +44 (0)20 7611 8722  
E [library@wellcomecollection.org](mailto:library@wellcomecollection.org)  
<https://wellcomecollection.org>

The Durex logo is displayed in white lowercase letters within a blue rounded rectangular border. The background of the entire cover is a collage of various images: a pink flower, blue eggs, a slice of orange, a strawberry, a close-up of a couple's faces, and a couple embracing in bed.

durex

a guide to  
**Condoms**





# The Durex Range

## Elite

- Extra lubrication for a more sensual experience
- Super thin for sensitivity
- Premium quality – the best in luxury
- Transparent coloured condoms
- Available in packs of 3, 12 and 18 condoms



## fetherlite

- Ultra Fine Condoms
- Meets desire for greater male sensitivity
- Spermicidally lubricated
- Transparent coloured condoms
- Available in packs of 3, 12 and 18 condoms



## Extra Safe

- Extra Safe is the heart of the Durex range
- Anatomically shaped for enhanced comfort
- Spermicidally lubricated
- Coral coloured condoms
- Available in packs of 3, 12 and 18 condoms



## Gossamer

- Improved non spermicidal lubrication
- More lubricious gentle natural feeling
- Committed experienced condom users
- Transparent coloured condoms
- Available in packs of 3 and 12 condoms



## Safe Play

- Highly focused, to have a relevance to a young audience
- Dual interpretation of name – safety combined with fun
- Spermicidally lubricated
- Transparent coloured condoms
- Available in packs of 3, 6 and 12 condoms



## ULTRAstrong

- Stronger condom for maximum security
- Plain ended condom
- Conforms to EN600 for strength claims
- Lubricious non-spermicidal lubricant
- Transparent coloured condoms
- Available in packs of 3 and 6 condoms



## Select

- A selection of ribbed, coloured and flavoured condoms
- Banana – Yellow condom
- Strawberry – Red condom
- Ice Mint – Blue condom
- Tangerine – Orange condom
- Ribbed – Coral coloured condom
- Puts the fun back into sex, by adding variety
- Lubricious non-spermicidal lubricant
- Available in packs of 3, 6 and 12 condoms



Hypo-Allergenic Natural Rubber Latex

CE  
0086



## IMPORTANT HOW TO USE A CONDOM



Only put the condom on when the penis is hard and erect. The fluid that is often released during the early stages of an erection can contain sperm (and STD organisms) so put the condom on before there is any contact between the penis and the vaginal area.



Tear along one side of the foil making sure that you don't rip the condom inside. Squeeze the opposite side of the foil and remove the condom carefully. Condoms are strong but they can be torn by sharp fingernails or jewellery.



Excess air trapped inside a condom could cause it to break so expel most of the air from the closed end as you put the condom on. Squeezing the closed end between your thumb and forefinger, place the condom over the erect penis making sure the roll is on the outside.



Using your other hand unroll the condom gently down the full length of the penis as far as it will go, still squeezing the closed end. Make sure the condom stays in place during sex; if it rolls up, roll it back into place immediately. If the condom comes off completely then a new condom must be used before sex continues.



Withdraw the penis soon after climaxing (cumming), holding the condom firmly in place at the base of the penis. Wait until the penis is completely withdrawn before removing the condom. Remember to keep both the penis and the condom clear from contact with your partner's body.



Wrap the condom in a tissue and dispose of it hygienically (not down the toilet).

*Remember, never use a condom more than once*

## HELPLINE

For immediate **FREE** information on sexual health matters phone **0800 665 544**. This service is funded and operated by the National Health Service. Lines are open 10am-5pm Monday to Friday.





All DUREX condoms are electronically tested and conform to British & the new single European condom standard. Look for the DUREX logo and DUREX quality seal of assurance.

The Durex feature list	Teat Ended  Plain Ended	Lubricants  Spermicidal  Non-spermicidal	Thickness	Colour	Shape	Pack Size
Fetherlite			Ultra Thin	Transparent		
Elite			Super Thin	Transparent		
Extra Safe			Regular	Coral		
Gossamer			Regular	Transparent		
Safe Play			Regular	Transparent		
Select			Regular	Various Colours		
Ultra Strong			Thick	Transparent		

Year:

Organisation: *Durex Info. Service for Sexual Health*

Commissioning Agency:

Main Theme/Approach: *Hetero / Guide to varieties of Durex condoms / how to use*

Media/Distribution: *pamphlet*

Endline:

Notes:

PRODUCED IN THE INFORMATION FOR SEXUAL HEALTH EDUCATION BY  
THE DUREX INFORMATION SERVICE FOR SEXUAL HEALTH  
LONDON INTERNATIONAL HOUSE  
BROXBORNE, HERTS EN10 6LN