

Durex Avanti : revoltionary non-latex condoms : please read this leaflet carefully / SSL International.

Contributors

SSL International

Publication/Creation

[2003?]

Persistent URL


<https://wellcomecollection.org/works/nsch9vuc>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).

**wellcome
collection**

Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

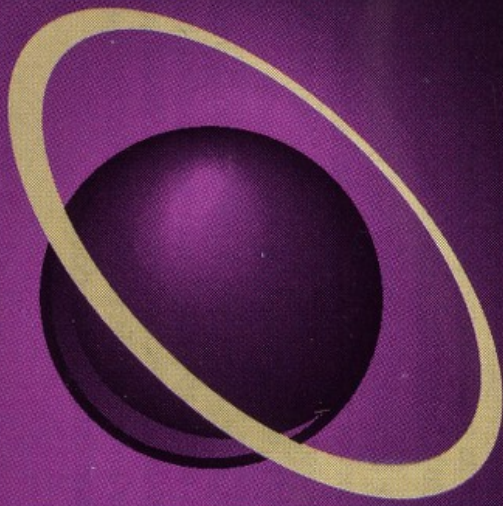

durex[®]

Avanti

Revolutionary non-latex condoms

PLEASE READ THIS LEAFLET CAREFULLY



Durex Avanti[™] condoms are the world's first male polyurethane condom. Made from a revolutionary new material, *Durex Avanti* condoms are much thinner than latex condoms, giving you the most natural feeling while still providing the protection you need.

Durex Avanti condoms are lubricated with a gentle non-spermicidal lubricant and have many qualities over standard natural rubber latex condoms.

- Non allergenic*
- Thinner and more sensitive
- Odourless
- Can be used with oil based lubricants as well as most water based lubricants
- Slightly larger for a more comfortable fit

Durex condoms are tried and trusted by millions of people world-wide. Indeed our commitment to quality has made *Durex* the world's leading condom brand.

* Non-Allergenic: *Durex Avanti* condoms contain none of the allergens found in natural rubber latex. Clinical testing confirms this. Should you experience any allergic reaction, discontinue use and contact your doctor.

HOW TO USE A CONDOM: Handle with care. Condoms can be torn by fingernails and sharp objects such as jewellery, zips and buckles.



Put the condom on when the penis is hard and erect, before there is any contact between the penis and your partner's body. Fluid released from the penis during the early stages of an erection can contain sperm and organisms of sexually transmitted infections.



Tear along one side of the foil making sure not to rip the condom inside. Squeeze the opposite side of the foil and remove the condom carefully.



Air trapped inside a condom could cause it to break. To avoid this, squeeze the closed end of the condom between your thumb and forefinger and place the condom over the erect penis. Be sure that the roll is on the outside.



Whilst still squeezing the closed end, use your other hand to unroll the condom gently down the full length of the penis as far as it will go. Make sure the condom stays in place during sex; if it rolls up during intercourse, roll it back to the base of the penis immediately. If the condom comes off completely, withdraw the penis and put on a new condom before intercourse continues.



Soon after ejaculation, withdraw the penis whilst it is still erect by holding the condom firmly in place at the base of the penis. Remove the condom only when the penis is fully withdrawn. Remember to keep both the penis and condom clear from contact with your partner's body.



Dispose of the used condom hygienically. Wrap the condom in a tissue and place it in a bin (do not flush it down the toilet).

NEVER USE A CONDOM MORE THAN ONCE

Store in a cool, dry place away from direct sunlight.



† Some ointments and gels applied to the genital area and macrogol (PEG)-based pessaries may damage this condom on contact. Always consult your pharmacist or doctor.

No method of contraception can provide 100% protection against pregnancy or the transmission of HIV and sexually transmitted infections.

If after intercourse you are concerned for whatever reason you or your partner may have become pregnant, contact your doctor immediately.

Non vaginal use of condoms may increase the risk of them slipping or being damaged.

If the condom slips or breaks, the users should contact a General Practitioner (GP), Family Planning Clinic, Women's Health Centre or Emergency Department for emergency contraception as soon as possible.

Durex manufacture a wide range of condoms specifically designed to offer a comprehensive choice to suit your needs and make condom use more pleasurable.

In addition to the unique *Durex Avanti* condoms contained in this pack the *Durex* range of natural rubber latex condoms are also available.

The spermicidally lubricated condoms in the *Durex* range are lubricated with nonoxynol 9. In laboratory tests, nonoxynol 9 has been shown to inactivate sperm on contact.

Some individuals may experience slight irritation to spermicides. If this should happen to you or your partner please consult a doctor. The *Durex* range also includes non-spermicidally lubricated condoms. Always check the expiry date on the foil. If this date is passed or if the foil is damaged do not use the condom inside.

Emergency contraception can be obtained from a GP, a family planning clinic, a young person's clinic, a Brook Advisory Centre and most genitourinary medicine (GUM) clinics.

USING AN ADDITIONAL LUBRICANT

Using an additional lubricant can make love-making more enjoyable. Unlike natural rubber latex condoms, *Durex Avanti* condoms may be used with oil based lubricants (such as baby oil and petroleum jelly) as well as water based lubricants.

SSL Australia Pty. Ltd.,
225 Beach Road,
Mordialloc VIC,
3195

Australia Customer Service
Freecall 1800 999 155

SSL New Zealand Ltd
PO BOX 100091,
Auckland,
NZ

Manufacturing site
SSL Manufacturing Cambridge,
Cambridge, UK.

On behalf of
SSL International Head Office
Canute Court,
Knutsford,
Cheshire,
WA16 0NL, UK.

UK Careline 0800 338 739
www.durex.com

Durex, *Avanti* and the *Durex Seal of Quality* are trademarks of the SSL group.



009974/1