Durex : tried and trusted by millions of people world-wide, Durex is the world's leading condom brand / SSL International.

Contributors

SSL International

Publication/Creation

[2003?]

Persistent URL

<https://wellcomecollection.org/works/na27jqwr>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).

**wellcome
collection**

Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

durex[®]

CE
0086

GB TRIED AND TRUSTED BY MILLIONS OF PEOPLE WORLD-WIDE,
DUREX IS THE WORLD'S LEADING CONDOM BRAND.

Do not use oil-based lubricants, such as petroleum jelly or oil-based pessaries. If an additional lubricant is required, ensure that the lubricant is compatible with latex condoms.

Non-vaginal use of condoms may increase the risk of them slipping or being damaged.

If used properly, latex condoms will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia, genital herpes, genital warts, gonorrhoea, hepatitis B and syphilis.

If after intercourse you are concerned for whatever reason you or your partner may have become pregnant, contact your doctor immediately.

Some individuals may experience sensitivity to latex products. If you are allergic to latex products please consult your doctor before use.

Some topical medicines may affect the efficacy of latex condoms on contact. Please consult your pharmacist/doctor on the compatibility of these products with condoms prior to use.

Some individuals may experience a slight irritation to spermicides. If this should happen to you, please consult a doctor. The Durex range also includes non-spermicidal condoms.

No method of contraception can provide 100% protection against pregnancy or the transmission of HIV and sexually transmitted infections.

For more information about Durex condoms or the range of Durex condoms available in your country, visit our website at www.durex.com call our Careline on 0800 338 739, or write to;

SSL International Head Office

Canute Court, Knutsford

Cheshire

WA16 0NL, UK

Durex and the Durex Seal of Quality device are trademarks of the SSL group.

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GB Put the condom on when the penis is erect, before there is any contact between the penis and your partner's body. Fluid released from the penis during the early stages of an erection can contain sperm and organisms that can cause STI's.

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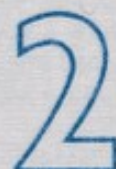
GB Tear along one side of the foil being sure not to rip the condom inside. Carefully remove the condom.

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3

GB Air trapped inside a condom could cause it to break. To avoid this, squeeze the closed end of the condom between your forefinger and thumb and place the condom over the erect penis. Be sure that the roll is on the outside.

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4

GB While still squeezing the closed end, use your other hand to unroll the condom gently down the full length of the penis.

Make sure the condom stays in place during sex; if it rolls up, roll it back into place immediately. If the condom comes off, withdraw the penis and put on a new condom before intercourse continues.

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