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ENGLISH (GB)

Store in a cool dry place, out of direct sunlight.

Instructions

1 - Open the packet by hand. Do not use sharp objects: scissors, knives, etc. take care not to damage the preservative with your fingernails. 2 - Apply the preservative on the uncovered glans, squeezing the reservoir of the preservative to drive out the air. 3 - Unroll the preservative to the bottom of the erect penis. 4 - Immediately after intercourse (ejaculation) and before the end of erection, withdraw, holding the preservative at its base. 5 - Dispose of the worn preservative in the bin after tying a knot in it. Never reuse the preservative. Each time you have intercourse use a new preservative.

Recommendations

The preservative must be placed on the erect penis before any contact between the penis and the partner's body to help prevent any sexually transmissible diseases or pregnancy. To reduce the risks of bursting the preservative, the use of gel is recommended; avoid using oily lubricants such as petroleum jelly and baby oil which damage the latex. Only use water based gels. Consult a doctor or chemist concerning the compatibility of prescribed preservatives with over-the-counter drugs intended to be applied in the vagina or in the penis.

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Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).