Durex / Reckitt Benckiser Healthcare (UK) Limited.

Contributors

Reckitt Benckiser Healthcare (UK) Limited

Publication/Creation

2002.

Persistent URL

<https://wellcomecollection.org/works/hzmwr9zp>

License and attribution

Conditions of use: it is possible this item is protected by copyright and/or related rights. You are free to use this item in any way that is permitted by the copyright and related rights legislation that applies to your use. For other uses you need to obtain permission from the rights-holder(s).

Wellcome Collection
183 Euston Road
London NW1 2BE UK
T +44 (0)20 7611 8722
E library@wellcomecollection.org
<https://wellcomecollection.org>

durex®

GB USING DUREX CONDOMS

This leaflet contains important information. It is worth taking the time to read it carefully.

Sex is more fun when you're relaxed and confident. If you've taken time to find out how to use condoms, you'll be able to enjoy the experience without anxiety spoiling the moment.

Take care to only use lubricants that are recommended for use with condoms, such as the Durex Play lube range. Oil-based ones like petroleum jelly, baby oil and some pessaries can damage condoms. So can some topical medicines that are applied to the penis or vagina. If you're not sure about the effect of the medicines you're using on condoms, ask your doctor or pharmacist.

Durex condoms are dermatologically tested. If you feel discomfort or irritation while you're using a Durex condom, stop using it. If your symptoms carry on afterwards, see a doctor. Some people are sensitive to latex. If you think you might be sensitive or allergic to latex, get advice from your doctor. Durex Avanti Ultima Polyisoprene condoms are suitable for latex sensitive people.

Usage - If they are used properly, condoms help to reduce the risk of catching HIV (AIDS) and many sexually transmitted infections. For anal sex, you should use additional lubrication on the outside of the condom to help reduce the risk of the condom breaking or coming off. (Don't use Durex Tingle condoms for anal sex.) Always read the guidance on lube packaging before you use it. If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows.

No method of contraception can provide 100% protection against pregnancy or the transmission of HIV and STIs. If you are worried that you or your partner may have become pregnant or infected, contact your doctor immediately.

Use a condom only once.

Store in a cool dry place away from direct sunlight.

Durex condoms are of course about your safety and sexual wellbeing. But they're also about your pleasure and satisfaction. Our range of condoms and sexual wellbeing products means you can make each sexual experience exactly what you want it to be, from reassuringly safer, to sensual, to exciting and intense with new vibrations and sensations! To further enhance your sexual experience, try the Durex Play lube range.

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durex®

1

(GB) Either partner can put the condom on the erect penis during foreplay. Take care to do this before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections.

NB: Check the expiry date on the condom wrapper before you use it. Tear the wrapper open from the serrated edge and handle the condom carefully, as it can be damaged by fingernails and sharp objects like jewellery and body piercings.

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2

(GB) Check the roll is on the outside. If it's on the inside, the condom is inside out. Squeeze the teat end of the condom so that no air is trapped inside.

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3

(GB) Still squeezing the teat, place the condom on top of the penis and roll it down with your other hand. If it starts to roll back up during sex, roll it back down straight away. If it comes off, stop and put a new condom on.

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4

(GB) Soon after ejaculation, and while the penis is still erect, the condom should be held firmly in place at the base of the penis before withdrawal. Then simply take the condom off, wrap it in a tissue and put it in the bin. Please don't flush it down the toilet.

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Check www.durex.com for information about our products and how to order them, as well as advice on sexual wellbeing.

WORLD'S NO1 CONDOM BRAND

Durex is the world's leading condom brand with over 80 years of quality experience.

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Reckitt Benckiser Netherlands.
Postbus 721,
2130 AS Hoofddorp.
Tel. +31(0)23 - 558 4500.

Reckitt Benckiser Belgium N.V./S.A.
Researchdreef 20 Allée de la Recherche,
B-1070 Brussel/Bruxelles.

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Reckitt Benckiser (Switzerland) AG.
(CH-8304 Wallisellen-Zürich)
Switzerland

Reckitt Benckiser Healthcare UK Ltd.,
Dansom Lane, Hull, HU8 7DS, U.K.

ISO4074:2002
*ISO4074:2002 and
associated quality
marks apply only to
Durex latex condoms

8012622

Rev 01

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S.p.A. - Milano- Italia
Reckitt Benckiser Austria GmbH,