

## **Marvel Whirling Spray / Constantine & Jackson, Limited.**

### **Contributors**

Marvel Company  
Constantine & Jackson (Firm)

### **Publication/Creation**

London : Constantine & Jackson, Limited, [approximately 1900]

### **Persistent URL**

<https://wellcomecollection.org/works/bua6nhy9>

### **License and attribution**


This work has been identified as being free of known restrictions under copyright law, including all related and neighbouring rights and is being made available under the Creative Commons, Public Domain Mark.

You can copy, modify, distribute and perform the work, even for commercial purposes, without asking permission.



Wellcome Collection  
183 Euston Road  
London NW1 2BE UK  
T +44 (0)20 7611 8722  
E [library@wellcomecollection.org](mailto:library@wellcomecollection.org)  
<https://wellcomecollection.org>

# MARVEL

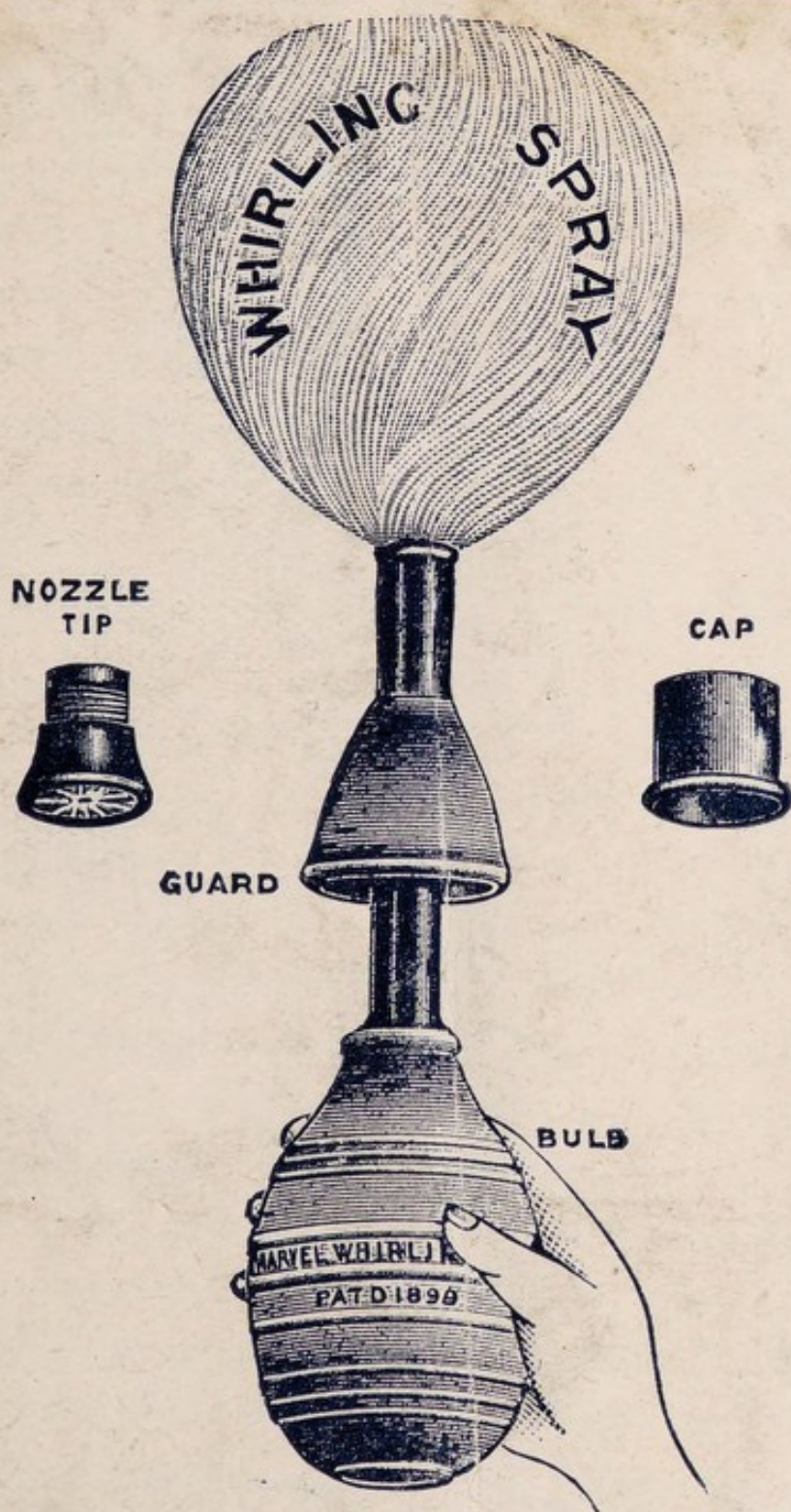

## WHIRLING SPRAY



**'MARVEL WHIRLING SPRAY'**

(TRADE MARK)

NEW WONDERFUL

# **LADIES' SYRINGE**

PATENTED 1899 IN THE UNITED STATES, GREAT BRITAIN, ETC.



**Price**

**15/= each**

*SIMPLE. ▯ SAFE. ▯ CONVENIENT.*

*IT CLEANSSES INSTANTLY.*





**T**HE "Marvel Whirling Spray" is the most wonderful improvement ever made in vaginal injection and suction syringes. It is entirely unlike the old style, clumsy solid-stream syringes, so long found inefficient and unsatisfactory. It is most simple, convenient, and effective, is always ready for use, has nothing to get out of order, and no cumbersome water-bags, tubing, valves, or shut-offs to cause delay and annoyance. The "Marvel" dilates and flushes the vaginal passage with a volume of whirling fluid which smooths out the folds and permits the injection to come in contact with its entire surface, instantly dissolving and washing out all secretions, etc.

The highly polished rubber vaginal pipe is five inches long. The whirling spray tip is removable for cleansing. The adjustable soft rubber guard closes the vagina, preventing the injection from escaping, and if desired causes its return by suction into the bulb when pressure on the latter is removed. In this way a douche can be taken in a moment without spilling the fluid. The rubber nozzle cap prevents leaking if the syringe is carried filled for use when travelling.

The Marvel syringe cannot throw a solid stream but discharges a volume of whirling spray, producing no shock, and is absolutely harmless.

Vaginal pipes that discharge a central or solid stream are condemned by physicians as exceed-



**ingly** dangerous. The fluid when thrown forward with force is liable to enter the uterus, causing thereby extreme pain, uterine colic, or alarming symptoms of nervous prostration and collapse. The "**Marvel**" discharges a half-pint of hollow whirling spray, which is more cleansing and effective than a gallon of injection used with any other syringe.

Fountain syringes with their many parts and attachments are inconvenient, and those which discharge small pin-like streams or a light film of water are **not effective**, for the reason that the small stream, when injected, reaches only that part lying against the nozzle; the course of the liquid is turned and follows back along the pipe and escapes, and its intended effect is lost. Only with the "**Marvel**" can the vaginal walls be distended by the **injection fluid**, which is **absolutely necessary** in order to reach and remove the **secretions in the folds**.

The soft rubber guard or plug not only prevents spilling, but when **hot injections** are used it protects the **sensitive external parts** from the heated liquid. Hot injections offer great relief for painful menstruation, and their use is advised during the interval between the menses. The most eminent physicians and specialists advocate the use of hot water as a very important agent in treating the troubles of women.

In the use of **hot-water injections** also for the



treatment of any vaginal disorder, no syringe can compare with the **Marvel**. Previous to the use of **medicated** injections, it is always advisable to cleanse the parts with warm soapsuds made by adding Castile soap to hot water, or dissolve a drachm of borax in a large cup of warm water. No assistance or attendant is required when using this syringe.

---

## **DIRECTIONS.**

**To fill syringe.** After removing rubber cap, compress the bulb as much as possible and place nozzle in fluid; relax pressure, allowing the bulb to expand, and it will **nearly fill** by suction. **Now exclude all air** by holding syringe with nozzle upright and compress bulb slightly until the fluid appears at tip. Now reverse position, still retaining pressure on bulb, place nozzle again in fluid, then relax pressure, and it will **completely fill**. **To thoroughly cleanse the parts**, insert the pipe and compress the bulb **sharply**, causing the whirling liquid to come in direct contact with those parts. **To withdraw injection by suction**, hold the guard firmly with two fingers of the left hand in the entrance to the vaginal canal, and then draw nozzle out to the entrance of the vagina, before permitting the bulb to expand.

When using **hot or medicated injections** the guard must be firmly held in the vaginal opening.



Press the bulb carefully, causing the vagina to partly fill. In a moment allow the fluid to return into the syringe. Now inject again and you will find the heated fluid has caused the parts to relax and permit of a larger injection; this may be retained within the parts as long as desired by keeping the bulb compressed. By relaxing the pressure, the injection, together with all secretions and discharges, will be instantly **drawn into the bulb by suction**. Eject fluid from syringe, cleanse same, and repeat the operation with fresh water. **To take a quick cleansing douche without suction**, remove guard, insert nozzle pipe, and inject a syringe or two of tepid water, allowing same to pass off in a douche pan or vessel. **After using syringe**, always rinse thoroughly, and stand on nozzle end until dry.

NOTE.—The nozzle tip may be unscrewed for sterilizing and thorough cleansing, also for filling the syringe without suction.

### **IMPORTANT.**

Before attempting to fill or use the syringe, read the directions over **twice** very carefully, and be sure that you follow them. A few moments' careful study of same, as also a little experimenting with the syringe, will enable you to use it intelligently, and to your entire satisfaction.

All who have tried the various vaginal syringes are fully aware of their imperfections or useless-



ness. The vagina being almost a closed aperture, every intelligent woman must perceive the utter impossibility of thoroughly cleansing without the use of this wonderful syringe, which dilates the passage with whirling liquid, instantly removing all secretions.

The "Marvel" is daily recommended by the most eminent Physicians and Gynæcologists for its simplicity and great convenience as the most effective and harmless vaginal douche ever invented.

---

## **AVOID CHEAP RUBBER GOODS**

The great competition in unprotected articles in the rubber line, especially syringes, the variety of which appears bewildering, compels the manufacturer to adulterate his rubber in order to lessen the cost of his goods, with the result, to the consumer, of leaky bags, broken tubing, and an accumulation of useless attachments at the expense of the disappointed purchaser. The syringe, even at the time of purchase, may not have been a practical or effective instrument. The new "Marvel Whirling Spray Syringe" is the finest piece of hard and soft rubber work of its class ever made

---

**IT IS FULLY WARRANTED.**



# **"WHIRLER" ANTISEPTIC TABLETS.**

**A Purifying, Healing, and  
Harmless Antiseptic, Deodorant, and  
Germicide.**

ESPECIALLY PREPARED FOR VAGINAL INJECTIONS

**Invaluable in Leucorrhœa or Whites,  
inflamed condition of the vagina, and  
as a solvent of all secretions and dis-  
charges.**

ESPECIALLY EFFICACIOUS AS  
AN EVERY-DAY INJECTION.

---

## **DIRECTIONS.**

**D**ISSOLVE one tablet in a half-pint of hot water, inject with the **Marvel Whirling Spray Syringe**, allowing the solution to remain in the vagina for about one minute, holding it by pressure on the bulb and soft rubber guard; by relaxing the pressure the injection, together with all secretions and discharges (dissolved), will be removed.

For *Leucorrhœa*, *Vaginal Weakness*, or offensive discharges, and to promote regularity in the menstrual period, the vagina should be thoroughly douched at least once a day. *For this purpose* one half a tablet to one pint of hot water will be sufficient; cleanliness is an absolute necessity to perfect health.

**Price per bottle, 100 tablets, 8/6**



# A VITALIZING TONIC AND RESTORATIVE TO THE SYSTEM.

---

## “WHIRLER”

### Viburnum Vitalizing Tablets.

---

**A** VALUABLE therapeutic remedy for debility in the genito-urinary system, especially indicated in urethral inflammation, ovarian pains, irritable bladder, aching back, prostatic troubles, and presenility or too early decline of the physical powers so often found among statesmen, preachers, lawyers, writers, business men and society women where too much nervous force or energy is expended.

“Whirler” Viburnum Tablets are a Vitalizing Tonic to the system of either sex. They act quickly and in a few days produce a feeling of buoyancy, strength and vigour which is not obtained in any other way. “Whirler” Viburnum Vitalizing Tablets are a true tonic and restorative, and not a temporary stimulant. Their regular and continued use for some weeks produces surprising results to the patient.

---

Price per bottle, 5/6.



**W**E have hundreds of Testimonials from the medical profession, and from ladies in all parts of the world, attesting to the peculiar merits of the Marvel Whirling Spray Syringe and the Whirler Antiseptic Tablets.

These Testimonials would convince the most sceptical.

---

## A FEW SAMPLES OF OUR MANY TESTIMONIALS

---

MARVEL COMPANY,

GENTLEMEN,—For a long time I have looked for a syringe which would thoroughly distend the vaginal folds in order to cleanse each one from any deposit or formation of deleterious matter existing therein. I have given your Marvel Whirling Spray a severe test, and I am glad to say that it is exactly what I require in every case of vaginal irrigation, for I am now convinced that the Marvel Syringe will, through its action, permit no secretion to remain after following the directions as given. I always recommend the Marvel Syringe and the Whirler Antiseptic Tablets to my patients.

Very truly yours,

JOHN J. McMAHON, M.D.



WORCESTER.

MARVEL COMPANY,

DEAR SIRS,—I am very much pleased with the **Whirling Spray**, and would not part with it for anything. I have recommended it to all my friends.

Yours respectfully,

MRS. C. L. W.

---

MARVEL COMPANY,

GENTLEMEN,—I received your letter and booklet some time ago, and as I had been a trained nurse I knew the good of your **Marvel Syringe** at sight. I turned the booklet over to my husband, told him to read it and see what he thought of it. The result was that you received an order from him for your **Marvel Syringe**, and I have it and would be willing to recommend it to anybody. I did show it to one lady and gave her one of the booklets. So many would think it a high-priced syringe, but if they had nursed as long as I have and worried with so many cheap syringes they would know at once that it is not, for it will last for years; and then it is the **only syringe that does the work properly**, and I will recommend it whenever I have an opportunity. I am **perfectly satisfied with the results of the Marvel Syringe**.

MRS. M. B. T.



MARVEL COMPANY,

GENTLEMEN:—I have been using the Marvel Syringe I purchased of you in July, and in that time have learned to know what a really valuable article you have sold me. I have come to value mine to the extent of being unwilling to part with it for many times its cost if I were not able to procure another. I have shown it to a number of ladies, several of whom have already sent you orders. I enclose Postal Note for 8/6; please send me another bottle of the Antiseptic Tablets. I would not be without them or my Marvel Whirling Spray.

MRS. CLARA E. H.

---

*From a Boston, U.S., Physician.*

I have great pleasure in writing you about your new Vaginal Syringe. It is without doubt the best syringe on the market. I am constantly recommending it in my practice.

---

*From a well-known English Physician,  
May 12th, 1900.*

DEAR SIR:—I received the syringe, and think it an excellent appliance. I have mislaid invoice, but enclose P. O., which I believe is the correct amount. Kindly send receipt. Yours faithfully.



*From a well-known New York Physician.*

I have now used the **Marvel Whirling Spray Syringe** in my practice, and find that it possesses many advantages over all other Vaginal Syringes now in use. Not only is it the most convenient, but for thoroughness in cleansing and effectiveness in applying medicated and hot injections, the **Whirling Spray Syringe** has no equal. We daily recommend it to the patients in our institute and sanatorium.

---

MARVEL COMPANY,

GENTLEMEN, -- The package containing the Syringe, Tablets, etc., is received, and I thank you for your prompt attention.

Allow me to say that I have used all kinds and makes of syringes, but yours **certainly is the best**. It does all you claim for it, and more, and I consider the price I paid for it one of the **best investments I ever made**.

Yours truly, MRS. H. L. S.

---

MARVEL COMPANY,

GENTLEMEN, -- I purchased a **Marvel Whirling Spray Syringe** of you on July 6th, and have now accidentally broken the tip and would like to learn if you can furnish one separately, and the price of same.

My experience with the **Marvel Syringe** up to this time has been so satisfactory that I **am more than pleased**, and am very much inconvenienced without it; so please do not delay, but answer at once, and very greatly oblige me.

Yours truly, MRS. E. B.



MARVEL COMPANY,

DEAR SIRs:—Mrs. W—— handed me your little booklet and explained to me the “marvel-  
lous” success that she has had with the syringe that she purchased of you some months ago, and I enclose herewith Money Order for 30/-, for which please send me, by express, two—one is for my daughter in Lafayette. Kindly send at once and oblige, Yours truly,

MRS. A. E. G.

---

DEAR SIRs:—

I received your booklet and was much obliged for your kindness. I had been very busy and had not time to write. I had been using the old fountain syringe, and it did not seem to do me any good, no matter how often I used it. My husband then recommended the **Whirling Spray** and asked our doctor about it; he said he thought it would be just the thing, so I purchased one, and to my great delight it gave PERFECT SATISFACTION the first time I used it. Every woman should have one of them.

Yours respectfully, Mrs. F. H.

---

*A Trained Nurse writes:*

LONDON, S.E., *March*, 1910.

I have derived much benefit personally from using the Spray with medicated fluids and my medical adviser Dr. N—— is most delighted with the result obtained. Mine was the first Syringe he had seen of that kind and I am sure he has done all in his power to get other patients to use them.



MARVEL COMPANY,

DEAR SIRs,—My wife received your communication and booklet, and she purchased one of your Syringes here in one of our drug stores. SHE IS MUCH PLEASED WITH IT.

Yours truly, T. J. O.

---

WARWICK.

DEAR SIRs,—Please send me another bottle of Antiseptic Tablets. Your Marvel Spray is indeed a “Marvel.”

Yours truly, MRS. C.

---

CATFORD, S.E.


MARVEL COMPANY,

GENTLEMEN,—Some time ago I purchased one of your **Marvel Whirling Sprays**, and I am writing to say how greatly pleased I am with it. I have used it regularly ever since I bought it, and I am more and more pleased with it the more I use it. It is just splendid, and by far the best Spray I have ever seen. I would not be without it for anything.

Yours truly, MRS. M.

---

## **Caution.**

This Syringe is broadly protected by Patents.   
The GENUINE bears the Trade Mark “Marvel Whirling Spray.” It is for sale by all Chemists, Surgical and Rubber Goods Houses everywhere.

**PRICE 15/.**



13/215  
71  
**M**ANUFACTURED in America for EXPORT  
under the personal direction of the  
Inventor and Patentee. ALL ORDERS  
to receive prompt attention should be addressed  
to

**CONSTANTINE & JACKSON,**  
**LIMITED,**

**Dealers in Surgical and  
India Rubber Appliances**

**City Pharmacy, 27, CHANCERY LANE,  
LONDON, W.C.**

**And at 73, RED LION STREET, HOLBORN.**

---

**WHOLESALE DEPOT:**

**THE MARVEL CO.,**  
**11, QUEEN VICTORIA STREET, LONDON, E.C.**

---

**PATENTED IN GREAT BRITAIN, JANUARY 3RD, 1899.**